- GB** **BALANCE indoor rollator**
- D** **BALANCE indoor Rollator**
- I** **Rollator da interni BALANCE**
- DK** **BALANCE indoor rollator**

Item nr. **312481** (red)
Art. Nr. **312481** (red)
Codice **312482** (grey)
Varenr. **312482** (grey)

User manual

MOBILEX A/S
Grønlandsvej 5
DK-8660 Skanderborg
www.mobilex.dk

Intended use of the device

This Mobilex product is a medical device which is intended to be used only as a walking assistant or chair by persons that are unable to stand, walk independently and who require additional stable support during walking. It gives for such persons the possibility to fulfil most of their regular, daily activities without the assistance of a helper. **Any other use is prohibited.**

Indications / contraindications

The rollator is indicated for use as walking and standing support by persons who suffer from different diseases in the human motion system. The variety of such diseases is very wide but the symptoms are similar: people who because of their health status are unable to walk or stand are indicated to use rollators. The final decision to choose the correct type of device depends on the condition of the individual person and should always be taken by a professional physician or physical therapist.

Safety requirements

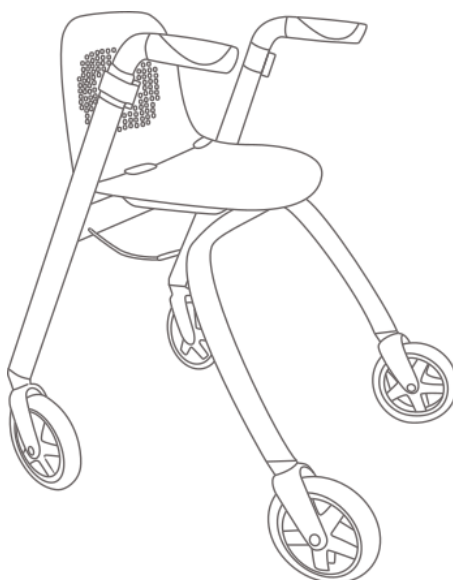
- Only use the product in flawless condition.
- If defects or errors are detected you must immediately contact the dealer.
- Follow the information on the product label.
- Only use the product for the described purpose.
- Avoid making constructive changes on the device, unless you have the manufacturer's written acceptance for such modifications.
- During different adjustments which are possible on the device the user must take care not to squeeze the fingers or other body parts between two moving elements of the device
- The device must only be used indoors on a flat and stable and surface.
- All wheels **MUST** be in contact with the floor at **ALL TIMES** during use. This will ensure that the rollator is properly balanced and should avoid accidents.
- When using the rollator in a stationary position, the hand brakes **MUST** be locked.
- The device must not be loaded with more than the max load value described in Technical Data and on the product's label.
- Do not lean backwards. The backrest is only designed to offer support while seated and does not prevent the user from falling backwards if the whole body weight is leaned backwards.

Product description

The Mobilex rollator "Balance" has a powder coated aluminium frame. It offers great manoeuvrability, stability and it has height adjustable push handles that can be fitted to the user without tools.

The rollator is supplied in a carton with the following content:

- ✓ *Main frame with wheels and seat*
- ✓ *This user manual*

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
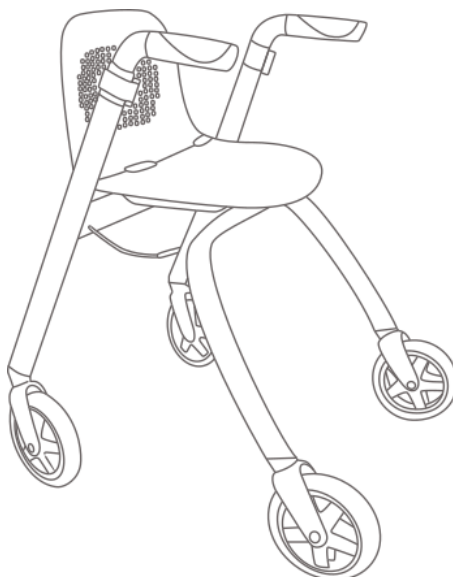
1

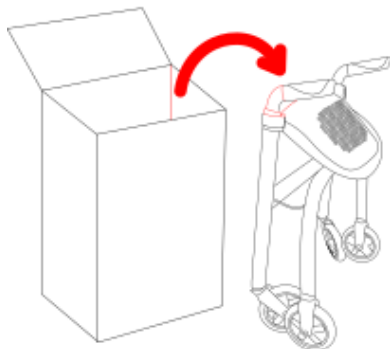
Take the rollator out of the carton and place it upright on the floor

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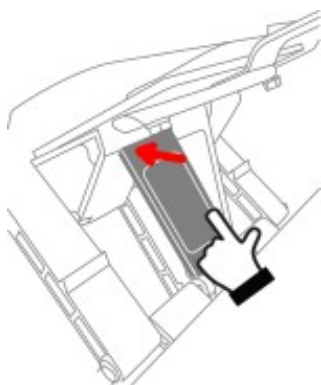2


Gently press the paddle under the seat in

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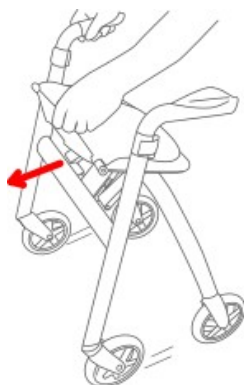
Rul forhjulene fremad

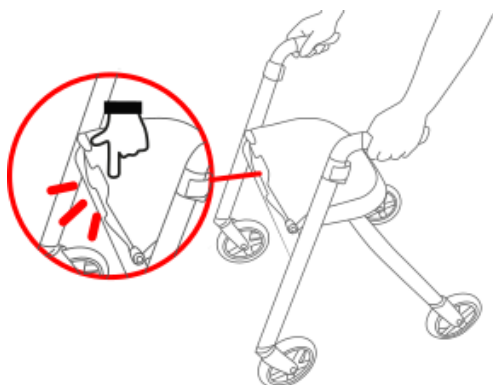4


Push down the seat to lock the frame until you hear a "click" sound. Attention! Make sure that the rollator is fully open before use !

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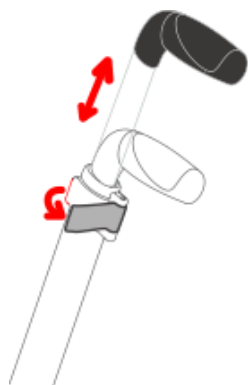5

To adjust the push handles in height:

- open the clip
- adjust the height of the handle
- close the clip

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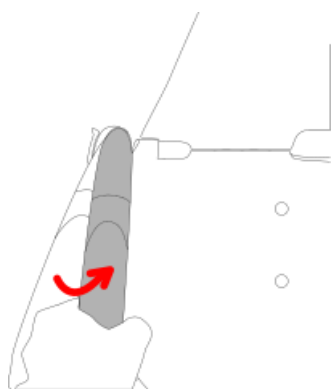
6

"GO" mode (brakes off)

to release the brakes push the handles inward and hold it there

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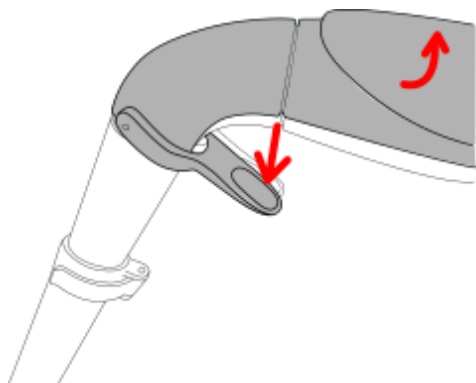
7

Fixed "GO" mode

push the handles inward and push down the lever to lock the handles in place. Please note that when the lever is down, the handle height is not adjustable.

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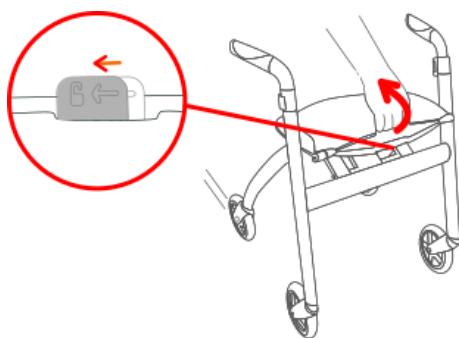
8

To fold the rollator:

- release the auto-lock by pushing the latch to left and hold
- then lift up the seat

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Guarantee terms

Mobilex A/S offers 2 years of warranty for damages caused by errors in production or material errors. The warranty is not valid by not-intended repair or usage of the product. Parts subject to normal wear are not covered by the warranty, unless wear is caused by a manufacturing fault. If product changes are made without our written acceptance the CE-marking and product warranty will not be valid. In the event of product damages covered by this warranty, please notify your dealer or Mobilex A/S directly. The warranty does not cover transport costs and does not include compensation for personal injuries or non-use of the product during repair. The warranty does not cover damages due to the user manual not being followed.

Declaration of Conformity (CE)

The company Mobilex A/S, Grønlandsvej 5, 8660 Skanderborg, Denmark declares that the product

Mobilex "BALANCE" indoor rollator (item no. 312481 and 312482)

is designed in accordance with the technical documentation and made of which conforms to the following basic requirements: **EU Council Directive 93/42/EEC for medical equipment**

Lars Nygaard
Managing Director
Mobilex A/S


.....
signature

Skanderborg, the 2018-09-03

Technical data

Mobilex Rollator Balance	Red Item no. 312481	Grey Item no. 312482
Height of the push handles (adjustable)	74,4 - 88,5 cm	
Size of the folded rollator (H x W x D)	80,7 x 62,7 x 26,0 cm	
Width	62,7 cm	
Length (depth) opened / folded	62,8 / 26,0 cm	
Weight	8,08 kg	
Max. load (rollator / bag)	120 kg	
Brakes	2 lockable with parking function	
Wheel sizes	Front and back: 6,4" (161 x 30 mm)	

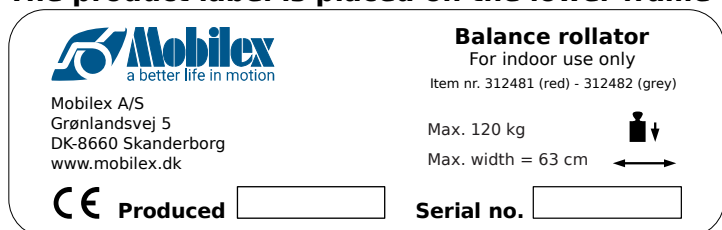The product label is placed on the lower frame

Example of product label (not original)