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ROLLATOR XXL ENORM & ENORM+

G-233 | G-234

CARELINE.de

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USER MANUAL (EN)

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USER MANUAL ROLLATOR XXL ENORM & ENORM+

The user manual has been written to enable you, the customer, to enjoy the full benefits of the product. Please read this manual carefully before assembling the product and using it. The contents of the manual are as follows:

- A. PRODUCT INFORMATION
- B. ASSEMBLY
- C. SAFETY INFORMATION
- D. OPERATING INSTRUCTIONSE. MAINTENENCE
- E. MAINTENANCE

PRODUCT CHANGES

The products shown in this manual by way of photographs and diagrams as well as products specifications may vary from time to time. We advise purchasers to check or any changes before a purchase is made.

A. PRODUCT INFORMATION - G-234 (ENORM)

overall height		min 91 cm, max 102 cm
overall length		70,5 cm
overall width		62 cm
seat depth		33 cm
seating width		45 cm
seat height		59 cm
folded width		24,5 cm
castor size		20 cm
unit weight		9 kg
weight capacity		200 kg

A. PRODUCT INFORMATION - G-233 (ENORM+)

overall height		min 87 cm, max 99 cm
overall length		70.5 cm
overall width		71cm
seat depth		35 cm
seating width		56 cm
seat height		55 cm
folded width		24.5 cm
castor size		20 cm
unit weight		9.2 kg
weight capacity		200 kg

B. ASSEMBLY - G-233, G-234

PARTS

- 1 x Unit (Bag, wheels and seat already attached)
- 1 x Backrest
- 2 x Handles

ASSEMBLY

1. Place handles into the frame (as shown below). Depress locking pins on the handles to allow the pin system to slide into the frame. Select desired height.
2. Depress locking pins on the backrest and slide into fixing provided on the upper cross-bar of the frame. Ensure that the locking pins have engaged properly into the holes and the backrest is secured.

C. SAFETY INFORMATION

1. If the walker is damaged or defective, it **MUST NOT BE USED** until it has been repaired.
2. The rollator is intended for indoor and outdoor use.
3. The walker must not be loaded with more than the specified maximum user weight.
4. The intended user must possess the necessary physical, visual and cognitive skills to operate the rollator safely.
5. **DO NOT** stand on this product.

USER MANUAL ROLLATOR XXL ENORM & ENORM+

D. OPERATING INSTRUCTIONS

1. (Pin System) To adjust height of handles. Depress the pins and adjust higher (pulling) or lower (pushing) until pins click into intended position securely. Ensure both handles are adjusted to the same height.
2. To apply brakes in motion, squeeze the brake levers located under the handgrips.
3. To secure walker in a stationary position, push down on brake handles until they lock into place.
4. When stationary and seated, brakes must always be applied.
5. To fold your Walker for storage/transportation, lift the seat to reveal handle on the inside. Pull the handle towards you to allow the unit to fold.

E. MAINTENANCE AND INSPECTION

It is important to carry out the following checklist at these commended intervals to ensure the product is well maintained. Identification of any irregularities should be notified to the appropriate sources and acted upon promptly.

	WEEKLY	MONTHLY	QUARTERLY
Wheels	X		
Knobs	X		
Handgrips		X	
Brake Cables	X		
Brake Lever		X	
Bolt and Nuts	X		
Bag			X
Bearings			X
Pins		X	

*It is highly recommended to have your Walker fully serviced at least once a year. Failing to do so will affect the performance of the product.

BRAKE ADJUSTMENT

To adjust the sensitivity of the brakes, pull the brake adjustment screw up to release the locking nut, then turn the locking nut to desired brake sensitivity. Turn locking nut clockwise to increase brake sensitivity. Turn locking nut anti-clockwise to reduce brake sensitivity.

Over adjustment of the brake will make the brake very difficult to engage. Under adjustment of the brake may cause the brake pad not to engage at all.

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CARELINE GmbH & Co. KG
CARELINE B.V.